

Customer:

Reference:

SPRAY GUN RINSER

PRODUCT DESCRIPTION

Spray gun rinse unit, for connection to water or compressed air, to be used for assisted cleaning of pipettes, catheters, cannulas, syringes, etc. The spray-gun is easy to operate with one (1) hand, enabling the operator to find the best angle for rinsing. The heat-insulated gun grip provides extra safety. Water/air pressure is released, regulated and fully controlled by the spray-gun trigger (adapted to a 1/2" connection). The spray-gun includes tubing and different tips and nozzles for various cleaning purposes. All appliances are stored within easy reach on the special-mounted rack. The tips are easily locked to the spray gun by a safety cone.

APPLICATION

Designed for use within soiled areas in reprocessing units, e.g. Central Sterile Processing/CSSD and the OR/point-of-use Sterile Processing/TSSU. Especially suitable for fine-mesh instruments.

KEY FEATURES

- Multi-purpose
- Flexible
- Quick and powerful

STANDARDS & CODES

Each spray gun is built in accordance with the following international safety norm:

- ISO 9001 (quality management systems)



PRINCIPLE OF OPERATION

Connect the spray gun hose to a water or compressed air source. Max water/air pressure 6 bar. In order to activate the spray gun, press the lever on the spray gun handle.

Equip the spray gun with a tip or nozzle if available.

Examples for usage of tips and nozzles:

- 1) catheters and small pipes
- 2) drainage tubing
- 3) spray jet for rapid instrument cleaning
- 4) syringes and cannulas with Record cone
- 5) measuring and blood pipettes
- 6) syringes and cannulas with Luer cone
- 3) spray jet for rapid instrument cleaning
- 7) bottles and Erlenmeyer flasks
- 8) waterjet pump for suction cleaning

TESTING

All spray gun rinsers have been tested according to Getinge demands.

TECHNICAL DATA

Tick your selection.

Art. No.	<input type="checkbox"/>	Product name	Number of nozzles included	Rack
470259001	<input type="checkbox"/>	Spray gun rinser	8	Included
470259003		Gun with hose without accessories	-	-