

XOButton Measurement Chart



XO Button[™] Sizing:

The appropriate XO Button[™] loop length is chosen based upon the desired length of graft in the femoral socket. The loop length is the difference between the AC length and the desired length of graft in the socket.

XO Button^M Loop Length = AC Length - Desired Graft in Socket Femoral Socket Length - Desired Graft in Socket ≥ 15 mm

Aperture to Cortex Length	Recommended Socket Length	Desired Length of Graft in Socket	XO Button [™] Loop Length = AC Length - Desired Graft in Socket
30mm	25mm	15mm	15mm = 30mm - 15mm
35mm	30mm	20mm	15mm = 35mm - 20mm
40mm and Greater	35mm	20mm to 25mm	20mm = 40mm - 20mm

©2007 Linvatec Corporation, a Subsidiary of ConMed Corporation COS 1050 Rev 1