

Ultra Series[®] Dual Purpose Shaver Blades

Resect tissue and debride bone conveniently with one shaver blade.



D4000 CONSOLE







To learn more about these and other innovative products, call **800-448-6506** or visit **ConMed.com**.

NEW



ULTRATIGER BLADE



ULTRAFRR BLADE



ULTRAGATOR BLADE

Dual Purpose with a Variety of Features

Designed to be more convenient and cost-effective, Ultra Series[™] Dual Purpose Shaver Blades can both resect tissue and debride bone. They feature a stronger, more robust cutting tip, longer region of lubrication and optimized dimensions. In addition, when powered by the D4000 Arthro Power Console/D4000A Arthro Power Console with Irrigation Pump, a Turbo feature enables you to toggle between oscillate and forward modes.

Stronger Inner Cutting Tip

A stronger inner cutting tip material and patented XtraSharp[™] technology provide superior cutting performance. This unique feature helps facilitate the process of debriding bone

Longer Shrink

A longer area of shrink, or lubrication, helps the Ultra Series Dual Purpose Shaver Blades operate at greater speeds.

Optimized Dimensions

A larger window allows the intake of both soft tissue and bone which can help procedures run more efficiently.

Color Hub

Color-coded hubs denote different blade styles.

The UltraTIGER[™] Excellent for general soft tissue resection and bone debridement.

The UltraFRR[™] Ideal for aggressive bone debridement and general tissue resection.

The UltraGATOR[™] Excellent for aggressive tissue resection and general bone debridement.



Greater Performance

at Your Fingertips

Longer shrink

To help streamline procedures, a Turbo feature toggles between tissue resection and bone debridement with the touch of a button. Simply press and hold the ON button and the handpiece moves from oscillate to the preset forward speed. The fact that you don't have to change shaver blades between tissue resection and bone debridement enables your procedures to be more efficient.

Providing Better Control

During Arthroscopic Procedures

Ultra Series[™] Dual Purpose Shaver Blades and Turbo feature work exclusively with the D4000/D4000A. The console can be preset to a specifically desired speed, which helps provide a more customizable approach to each unique arthroscopic procedure. Preset speeds from 0 to 4,000 rpms in oscillate and up to 12,000 rpms in forward/reverse.



ARTHROSCOPIC RESECTION

TURBO MODE

Color-coded hub



525 French Road, Utica, NY 13502

ULTRA SERIES" DUAL PURPOSE SHAVER BLADES

Advancing the Future of Minimally Invasive and Orthopaedic Surgery. **Together.**

Ordering Information

Description	Working Length	Catalog Number
3.5 mm UltraGator [™]	13cm	DPS-C001
4.2mm UltraGator [™]	13cm	DPS-C002
5.5mm UltraGator	13cm	DPS-C003
3.5 mm UltraTiger [™]	13cm	DPS-C004
4.2mm UltraTiger [™]	13cm	DPS-C005
3.5 mm UltraFRR™	13cm	DPS-C009
4.2mm UltraFRR™	13cm	DPS-C010
5.5mm UltraFRR	13cm	DPS-C011
3.5mm Ultracut®	13cm	35ULT-ST-ZZ
4.2mm Ultracut	13cm	C9405A
5.5mm Ultracut	13cm	C9415
4.2mm Ultracut Pre-Bent	13cm	42ULT-ZZ-MT
4.2mm Ultracut Extended Length	19cm	EL9405

Local 727-392-6464 Toll Free 800-237-0169 ConMed.com customer_service@conmed.com